

Product Description


Central Venous Catheter is a Radio Opaque catheter placed mainly in the subclavian or jugular veins down to the right Auricle as a short-term Channeling access to the central administration of drugs, chemotherapy and central infusion of fluids and transfusion of blood.


Central Venous Catheter have become an indispensable tool in anesthesia, critical care and emergency medicine. Their use is a standard element of Modern Medical Practices.

Saudi Mais Co. for Medical Products

Third Industrial Area, Al-Kharj Road, Street No.256 P.O Box: 3900, Riyadh 14335-7599 Saudi Arabia

Central Venous Catheter Set

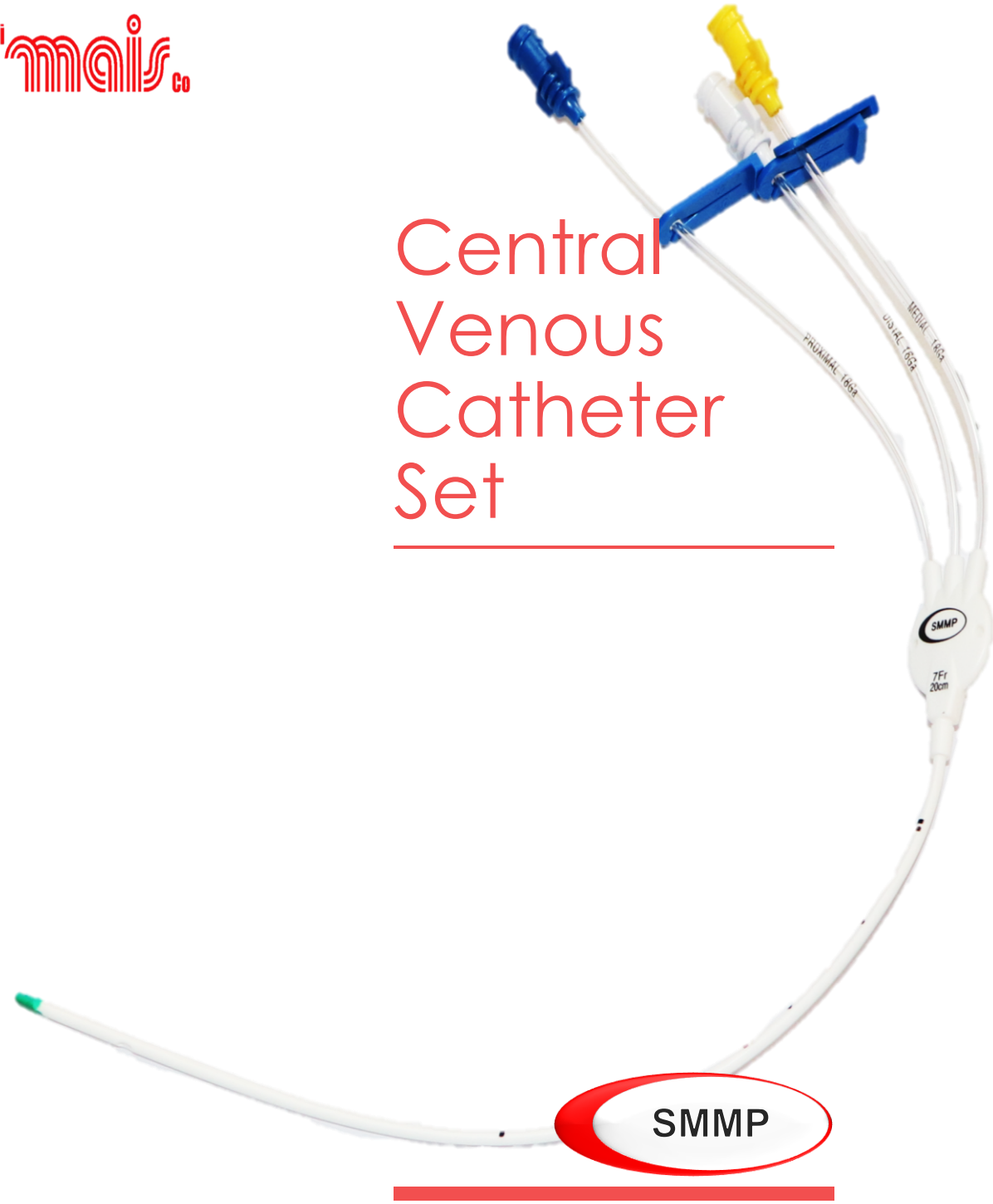

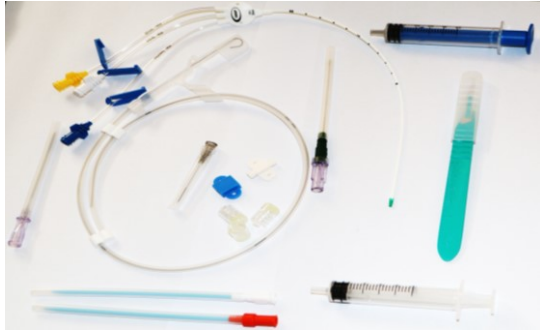

Intended Use:

The Central Venous Catheters are intended for vascular access infusion and withdrawal of blood, blood products and fluids, Central venous blood pressure monitoring (CVP) and Central Venous Blood Sampling

It may be used for administration of irritant drugs (e.g.: Catechol Amines, Chemotherapeutics).

Technical Features:

- Catheter Indwelling
- Made of Medical Grade Material.
- Radio Opaque
- Coated with thrombo resistant substance which facilitate smooth insertion and prevents potential.
- Adhesion of Bacteria or thrombus formation.
- Graduated Accurate Length
- Color Coded Luer Connectors
- Guide Wire-Spring Wire of High-quality SS “J” Shaped or “Straight”
- Injection Sealing Cap for Protection
- Scalpel #11
- Adjustable Fastening Pri. And Sec. fixation devices for Secure Connection.

Size & Specifications

Cat.Ref	Type	Size FR	Length cm	G.W cm	Dilator	Introducer Needle	Other Components	Optional Components					
623-0405	CVC D/L	04	04	0.018" x 50	5.5F x 6cm	20	5ml Syringe: 1 Pcs 5ml Introducer Syringe: 1Pcs Hypodermic Needle: 2Pcs Slide Clamps: 2Pcs Movable Clamp: 1Set Injection Sealing Cap: 2Pcs	3-Way Stopcock: 2Pcs I.V Cannula: 1Pcs (G:22)					
623-0460	CVC D/L	04	06										
623-0408	CVC D/L	04	08										
623-0413	CVC D/L	04	13					0.021" x 50cm	6F x 7cm		3-Way Stopcock: 2Pcs I.V Cannula: 1Pcs (G:18)		
623-0560	CVC D/L	05	06										
623-0508	CVC D/L	05	08										
623-0515	CVC D/L	05	15	0.021" x 50cm	6.5F x 10cm	20	5ml Syringe: 1 Pcs 5ml Introducer Syringe: 1Pcs Hypodermic Needle: 2Pcs Slide Clamps: 3Pcs Movable Clamp: 1Set Injection Sealing Cap: 3Pcs	3-Way Stopcock: 3Pcs I.V Cannula: 1Pcs (G:20)					
623-5508	CVC D/L	5.5	08										
623-5513	CVC D/L	5.5	13										
623-0715	CVC D/L	07	15										
623-0720	CVC D/L	07	20						0.035" x 60cm				
623-0730	CVC D/L	07	30						0.035" x 70cm				
625-0515	CVC T/L	05	15						0.035" x 60cm	9F x 10cm	18G	Slide Clamps: 4Pcs Sealing Cap: 4Pcs other components same as above	3-Way Stopcock: 3Pcs I.V Cannula: 1Pcs (G:20)
625-5505	CVC T/L	5.5	05										
625-5506	CVC T/L	5.5	06										
625-5508	CVC T/L	5.5	08										
625-5513	CVC T/L	5.5	13										
625-0715	CVC T/L	07	15										
625-0720	CVC T/L	07	20										
625-0730	CVC T/L	07	30										
625-7520	CVC T/L	7.5	20	0.035" x 60cm									
625-0820	CVC T/L	08	20										
627-7520	CVC Q/L	7.5	20	0.035" x 60cm	9F x 10cm	18G	Slide Clamps: 4Pcs Sealing Cap: 4Pcs other components same as above	3-Way Stopcock: 3Pcs I.V Cannula: 1Pcs (G:20)					

Contraindications:

1. Never implant catheter in a thrombosed vessel.
2. Infection or cut wound around the puncture area.
3. Dysfunction of blood coagulation thrombus formation.

Warnings:

- Only use by professional Healthcare practitioner.
- Do not exceed recommended capacities.
- Designed for Single use Only.
- Sterility will not be guaranteed if the package is damaged / opened.
- Authenticate the integrity of all parts and packaging before use. If there is damage, please do not use this product.
- Discontinue the use if it causes discomfort and consult with physician.

Contact Us

Saudi Mais Co. for Medical Products
Third industrial Area, Street No.: 256
P.O Box:3900, Riyadh 14335 - 7599

+966 11 265 0184
medical@mais.com.sa

Visit us on the Web:
www.maisco.com.sa